

Exporting DICOM files from Sirona SIDEXIS 4

1

Create a New Folder with the Patient's name

- Right-click anywhere on your desktop where there isn't an icon.
- Select **New > Folder**, and name the folder using the **patient's full name**.

2

Open the CBCT Scan

Launch SIDEXIS 4 and open the desired CBCT scan for the patient.

3

Access the Export Menu

In the top toolbar, click the "Output" tab on the right. Then select "DICOM Exam Export."

4

A "DICOM Exam Export" window will appear.

5

Adjust Settings

- Under the **Settings** tab:
- Ensure that only "Include main volume" is checked.
- Uncheck all other options.

6

Switch to the Export tab:

Make sure "Anonymization" is unchecked.

7

Select Export Location

- Click the "..." button next to the export directory field.
- Navigate to the folder you created in Step 1.
- Click "OK" to begin exporting the DICOM files.

8

Compress the Folder

(Do not skip this step!)

- On the desktop, Right-click on the patient's folder now with DICOM files.
- Select "Send to" → "Compressed (zipped) folder."
- A zipped version of the folder will be created, typically marked with a zipper icon or a blue "Z".

9

Upload the Zipped Folder

- On IDR portal, navigate to the **IDR interpretation request form**.
- Upload the zipped folder as part of your case submission.